LUMINA™ Post-op dressing

en

Description:

LUMINA Post-op dressing consists of a thin film laminate backing with a non-latex adhesive and a non-adherent pad. The dressing is breathable, allowing good oxygen and moisture vapour exchange. It is waterproof and impermeable to liquids and bacteria. An intact dressing protects the site from outside contamination. The dressing shall be applied and removed by a healthcare professional. Before removal the adhesion is reduced by illuminating the dressing with the UV-A lamp or by exposing the dressing for sunlight.

Indications:

LUMINA Post-op dressing can be used for covering acute wounds such as cuts, superficial and partial thickness burns, abrasions, IV catheter sites, and surgical incisions. It can also be used for covering low exuding superficial and partial thickness wounds including pressure ulcers, venous ulcers and diabetic foot ulcers.

Contraindications:

Sensitivity towards UV-A radiation.
Allergy / sensitivity towards polyurethane adhesives.

Warnings:

Do not use the dressing as a replacement for sutures and other primary wound closure methods.

Do not look directly into the UV-A lamp when it is turned on.

Do not expose the dressing to light during use, not even indoor light.

If the sterile packaging is damaged, the dressing must not be used but discharged.

Precautions:

- Stop any bleeding at the site before applying the dressing.
- Do not stretch the dressing during application as tension can cause skin trauma.
- Make sure the skin is clean, free from soap residue and lotion, and allowed to dry thoroughly before applying the dressing to prevent skin irritation and to ensure good adhesion.
- LUMINA Post-op dressings should not be re-sterilized.

Instructions for use:

Dressing selection:

Choose a dressing size large enough to provide at least two centimetres' margin which adheres to dry, healthy skin around the wound site.

Site preparation:

- Prepare the site according to institution protocol.
- Clipping of hair at the site may improve dressing adhesion. Shaving may reduce pain on dressing removal.
- Allow preps and protectants to dry completely before applying the dressing to prevent skin irritation and to ensure good adhesion.

Application:

- Open the package and take out the sterile dressing.
- Peel part of the paper liner from the dressing, exposing the adhesive surface (see figure 1).
- Centre the dressing over the application site.
- Press the dressing into place.
- Remove remaining parts of the liner from the dressing without stretching the dressing and press the dressing into place.
- Smooth the dressing from the centre toward the edges, using firm pressure to enhance adhesion.
- Cover the dressing from direct or indirect sunlight with clothes or bed clothes (see figure 2), to prevent unintentional reduction of adhesion.

Site care:

- The site should be observed for signs of infection or other complications. If infection is suspected, remove the dressing, inspect the site directly, and determine appropriate medical intervention. Infection may be signalled by fever, pain, redness, swelling, or unusual odour or discharge.
- Change the dressing according to institution protocol, or when the barrier properties have been compromised. For example, if the dressing is loose or punctured by sutures or other devices, it should be changed.

Removal:

Alternative A

- Select the UV-A lamp (see figure 1 below), direct it at the dressing border at a distance of 4-5 centimetres.
- Turn on the lamp and illuminate an area of the dressing border for 2-5 seconds.
- Move the UV-A lamp to illuminate the next area of the dressing border for 2-5 seconds.
- Repeat step 3 until the entire area of the dressing border has been illuminated (see figure 2 below).
- Turn off the UV-A lamp.
- Gently grasp an edge and slowly peel the dressing from the skin in the direction of hair growth (see figure 3 below). If adhesion level is still high, repeat step 1-6 with 5 seconds illumination per area.

Alternative B

- Expose the dressing to sunlight (see figure 1).
- Keep it in the sunlight for at least 1 minute (see figure 2).
- Gently grasp an edge and slowly peel the dressing from the skin in the direction of hair growth (see figure 3). If further adhesion reduction is required, repeat exposure to sunlight.

Shelf life and storage information:

For best results, store in a cool, dark and dry place. For shelf life, refer to the expiration date stated on each package.

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Lumina Adhesives AB